

iPad Pro Mounting System & iPad Pro Platform (Only)



Catalog No. 1587 & 1588

Enabling Devices

50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

The ultimate accessory!

Our terrific iPad Pro system is available in two versions: Mounted and iPad Pro Platform only. The Mounting version comes with a mounting platform, a heavy duty dual arm, and a super clamp. The mounting platform is designed for easy access to all of the iPad' Pros electronic ports. It also has two flexible cords to securely hold the iPad Pro. Viewing angle can be set either landscape or portrait. The heavy duty mounting arm expands from 14½" to 24" for maximum flexibility and reach. The clamp allows you to easily mount the system to a wheelchair or tabletop. The Mounting System works with full sized iPad Pro 12.9 inch screen (not iPad Pro 9.7 inch screen). It is also compatible with the iPad Pro case (#1586). Mounting platform is also available separately. Platform size: 10¾"L x 9½"W x 1½"H Weight 2 lbs.; Arm size: expands from 14½"L to 24"L. Mounting System Weight 4 ½ lbs.

1. **Mounting Arm Assembly:** Your mounting system consists of three main parts, an adjustable arm, iPad Pro mounting platform and a Super clamp. First you will need to assemble the 3 main parts. To attach the clamp to the arm, loosen the set screw on the clamp (see Fig. 1). Next depress the Silver Spring loaded locking pin, then push one of the ends of the mounting arm into the clamp(see Fig. 2) Please Note: you need to line up the flat part of the Mounting Arm end post with the set screw in the clamp. This will prevent the arm from spinning freely inside the clamp. Once this is done you can tighten the setscrew to lock the Mounting arm to the clamp.



Fig. 1



Fig. 2

2. **Clamping Options:** Our mounting clamp is designed for mounting to tubular (wheelchair, bedrail, walker, etc.) or flat surfaces such as a table top or desk. For flat surface mounting use the included wedge insert shoe stored in front of the locking knob on the clamp (see Fig.3) Loosen locking knob slightly to remove wedge, tighten knob once wedge has been removed.



Fig.3

3. At this point we recommend you place your mounting arm without the iPad Pro mounting platform and do a rough setup, you can fine tune your adjustments after you have placed the iPad platform on the arm. Do not force or try adjusting the arm without first loosening the set knobs for that portion of the arm. Each section of the arm has a set knob. Once you have the arm in the desired position you can tighten the knobs to lock the arm in place.

4. Once the arm is in position you can now place your iPad and platform on the arm. First back off the set knob on the iPad Pro platform until it is flush with the inside of the opening. This is necessary to allow the holder to be pushed all the way down on the mounting arm post. Once this is done you can tighten the set knob to lock the platform to the arm. Please Note: Make sure to back the set screw enough to allow the platform to be pushed all the way down on the post of the arm, failure to do so will cause the iPad to fall off the mounting arm and a fall could cause damage to your iPad. (See Fig. 4).

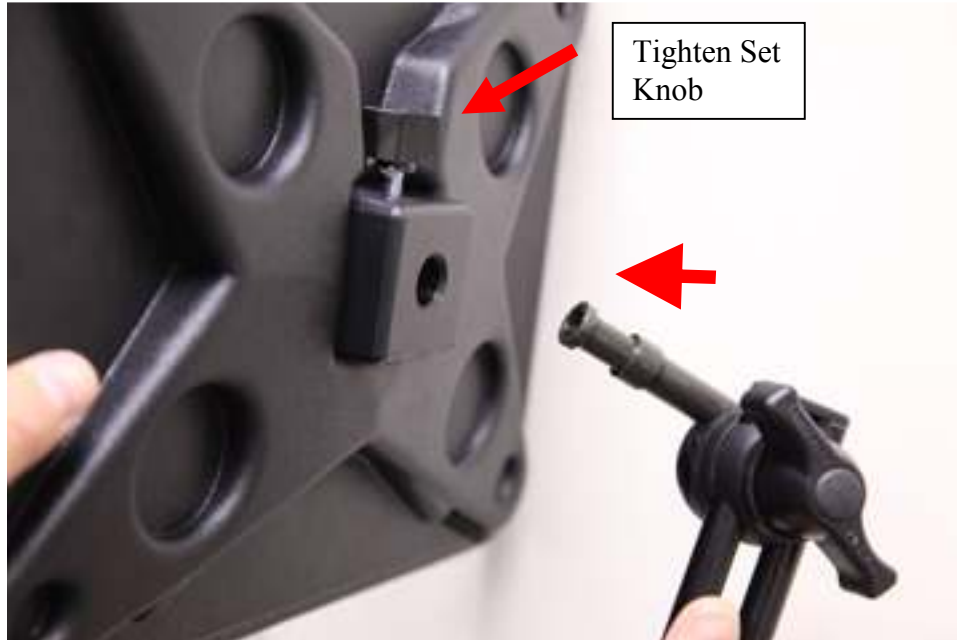


Fig.4

5. **Installing your iPad:** Now that you have your mounting arm assembled and adjusted you can now place your iPad Pro into the holder. Slip one side of your iPad Pro under the bungee cord holder, making sure you have all your ports lined up with the holder. Adjust and secure with the second bungee cord holder (See Fig. 5). The iPad Pro platform can be oriented in either the landscape or portrait position (See Fig. 6). This mount will hold any full size iPad Pro with or without a case. (We recommend our case Item No.1586 for the iPad Pro).



Fig. 5



Fig. 6

Video: For more detailed setup information please view our setup video at: www.enablingdevices.com and type in the Item no. 1587 in our search box and then click on the results to take you to the video.

Caution:

Do not over tighten the adjusting knobs, or try to move or adjust the mounting system after you have tightened the adjusting knobs. Doing so can damage the Mounting System and void your warranty.

Care of Unit:

The Mounting System can be wiped clean with any mild household multi-purpose cleaner and disinfectant. **Do not use abrasive cleaners**, as they will scratch the surface of the unit.

iPad™ is a trademark of Apple, Inc.

For Technical Support:

Call our Technical Service Department

Monday through Friday, 9 a.m. to 5 p.m. (EST)

1-800-TEC-TOYS or 1-800-832-8697

customer_support@enablingdevices.com