



(Photo 3)

Video: For more detailed setup information please view our **setup video** at: www.enablingdevices.com and type in the Item no. **1589** or **1556X** in our search box and then click on the results to take you to the video.

Care of Unit:

The Mounting System can be wiped clean with any mild household multi-purpose cleaner and disinfectant. **Do not use abrasive cleaners**, as they will scratch the surface of the unit.



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-832-8697
customer_support@enablingdevices.com

IPad Tabletop System #1589
IPad Table Top Base (Only) #1556X

USER'S GUIDE



50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

The ultimate accessory!

Our terrific mounting systems are available in two versions: Tabletop and Mounting. The Tabletop version comes with a mounting platform and a removable tabletop base. The Mounting version (#1556) comes with a mounting platform, a light duty mounting arm, and a clamp. The mounting platform is designed for easy access to all of the iPad's electronic ports. It also has two flexible cords to securely hold the iPad. The tabletop base holds your iPad at a perfect viewing angle and can be clamped. The light duty mounting arm expands from 14½" to 24" for maximum flexibility and reach. The clamp allows you to easily mount the system to a wheelchair or tabletop. Mounting Systems work with iPad 1, iPad 2 or iPad with retina display. It is also compatible with the iPad case (#1558). Removable tabletop base and mounting platform are also available separately. Platform size: 10¾"L x 9½"W x 1½"H; Arm size: expands from 14½"L to 24"L. **PLEASE NOTE: iPad not included.**

1. The table top base is held in place with 4 Phillips head screws. To install or remove the base, use a Phillips head screwdriver see photos. 1 & 2.



(Photo 1)



(Photo 2)

2. **Installing your iPad:** Now that you have your mounting arm assembled and adjusted you can now place your iPad into the holder. Slip one side of your iPad under the bungee cord holder, making sure you have all your ports lined up with the holder. Adjust and secure with the second bungee cord holder (**See photo 3 on back page**). This mount will hold all generations of the iPad with or without a case. (We recommend our case Item No.1555 for the iPad 1 and No. 1558 for iPad 2 or newer).
3. **Mounting Arm Use:** Your iPad Tabletop System can also be used with our **#1557** Light Duty Magic Arm System or our **#1631** Magic Arm Mounting System. Both sold separately.