
Busy Ball Popper #1593

USER'S GUIDE



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-832-8697

customer_support@enablingdevices.com



50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

Magically powered by air!

It's poppin', droppin' air-powered fun! Press the button or activate your capability switch and watch the 5 colorful balls pop up and then roll down the track. Silly sound effects and lively songs also play, adding to the sensory experience. Toy runs for approximately 20 seconds with each activation and has an on/off switch to save battery life. A clear dome keeps the balls in action. Great for increasing visual attention and tracking. Size: 13"L x 5¾"W x 18"H. Requires 4 D Batteries. Weight: 3½ lbs.

Operation:

1. The Busy Ball Popper requires four D batteries (Not included). Alkaline batteries are recommended because they last longer. Do not use rechargeable batteries because they supply lower voltage and the unit may not perform as well.
2. The battery compartment is located on the back of the unit. To remove the battery compartment cover, unscrew the small Phillips Head screw, then slide the cover back and then off. Install new batteries, being careful to observe proper (+) & (-) battery polarity. Replace cover and screw.
3. Turn the ON/OFF switch located on the back of the unit to the ON position. Connect the unit to any external capability switch via the 1/8" jack, located on the back of the unit. If you need to use a ¼" to 1/8" adapter for your switch, it must be a mono adapter, not stereo.
4. Make sure that the connections between the external capability switch and the adapter (if applicable) and the jack on the Busy Ball Popper is plugged in all the way. There should be no gaps.

5. Load the supplied balls into the ball slid. Begin play by activating your external capability switch. This will start the balls popping up into the clear dome and rolling down the track, while simultaneously playing silly sound effects and up to eight lively songs.
6. The Busy Ball Popper will continue playing while your switch continues to be activated. Once you release your switch, there will be a brief delay then all will stop.

Troubleshooting:

Problem: The Busy Ball Popper does not work.

Action #1: Check or replace the four D batteries with fresh new batteries. Never mix different brands or old and new batteries.

Action #2: Make sure the Busy Ball Popper is on a flat level surface and that nothing is restricting any of the balls. Use only the balls included with Busy Ball Popper, other balls will not work with this product.

Action #3: Try a different switch that you know works with other toys/devices to rule out the Busy Ball Popper as the source of the problem.

Care of Unit:

The Busy Ball Popper can be wiped clean with any household multi-purpose cleaner and disinfectant.

Do not submerge the unit, as it will damage the contents and the electrical components.

Do not use abrasive cleaners, as they will scratch the surface of the unit.