
Magical Light Show #1672

USER'S GUIDE



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-832-8697

customer_support@enablingdevices.com



50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

Lights, music, and vibration!

Press the yellow switch and feel subtle vibrations under your hand while the Magical Light Show begins. Provides great tactile awareness. Watch colored patterns against a black background and hear upbeat music. There is an external jack for your capability switch. Size: 10¼"L x 11"W x 11½"H. Requires 5 AA Batteries. Weight: 2¼ lbs.

1. The Magical Light Show requires 5 AA batteries (not included). Alkaline batteries must only be used (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. The battery compartments are located under the unit's base. Gently turn the unit over to reveal the black battery compartments. Carefully remove the small screw from each of the battery compartment covers using a Phillips head screwdriver, and then slide the battery compartment covers off. Install AA size batteries into each holder. When installing new batteries, always be careful to observe battery polarity. Replace battery covers and screws. Never mix old and new batteries together or different brands or types together.
2. Turn the ON/OFF/VOLUME knob clockwise, you will hear an audible click. Adjust to knob for desired volume level.
3. Simply press on the round yellow switch plate and you will be rewarded with spinning lights, music and vibration. Release pressure from the switch and the Lights, Music and Vibration will stop.
4. To use the Magical Light Show with a switch, plug, any Capability switch into the switch jack located on the side of the Magical Light Show.
5. Activate your switch and you will be rewarded with spinning lights, music and vibration. Release pressure from the switch and the Lights, Music and Vibration will stop.

Please Note: Not to be used with seizure prone individuals.

Troubleshooting:

Problem: Magical Light Show does not spin or light up.

Action #1: Check the Batteries for strength and for proper battery polarity. Replace if weak or dead.

Problem: Magical Light Show does not play music or vibrate.

Action: Check the Batteries for strength and for proper battery polarity. Replace if weak or dead.

Problem: Magical Light Show does not work at all.

Action: Check all Batteries for strength and for proper battery polarity. Replace if weak or dead.

Problem: Magical Light Show does not activate an external switch.

Action # 1: Make sure all connections are tight.

Action # 2: Check your switch with another device you know is working.

Action # 3: If an adapter is being used, it must be Mono, stereo adapters will not work.

Care of the Unit:

Magical Light Show can be wiped clean with any household multi-purpose, non-abrasive cleaner and disinfectant. We recommend Simple Green, which is a non toxic biodegradable all-purpose cleaner.

Do not use abrasive cleaners, as they will scratch the surface of the unit.

Do not submerge the unit, as it will damage the electrical components.