#### **Encourages surprising rewards!**

Bright activity center encourages therapeutic movements, increases finger isolation and grasping while providing tactile, auditory and visual stimulation. Pull on one of the two wiggle people and goofy cartoon sounds will play. Rotate the worm and set off a potpourri of lights, music and vibration in the dome. If you move the giant knob to the left or right, the user will hear more cartoon sounds. Pushing the green button plays a soothing tune. The wedge base can be easily removed so that the manipulator can be wall mounted (Hardware included). Size: 12"L x 14"W x 3"H. Requires 4 C and 4 AA Batteries. Weight: 4½ lbs.

#### Operation:

- 1. The Therapeutic Manipulator requires four C batteries (not included) and 4 AA batteries (not included). The battery compartments are located under the base of the unit. Carefully turn the unit over to remove the base stand. Simply pull the base stand off. Velcro holds the base stand in place. Next install 4 new C cell batteries, being careful to observe battery polarity. Alkaline batteries are recommended because they last longer. Do not use rechargeable batteries because they supply lower voltage and the unit may not perform as well. To install the 4 AA batteries, unscrew the Phillips head screw on the black battery case cover. Install the batteries the same way as the C batteries. Replace the battery case cover and screw.
- 2. Replace the wedge base stand.
- 3. The Therapeutic Manipulator is a somatosensory toy. Pull on either one of the two string people, to be rewarded with a zany sound. Moving the Inch Worm in a left to right motion, will cause the center glitter dome will vibrate, spin

its glitter, play a melody, and its surrounding lights will light up. Pressing and holding the green button on the lower left hand side, will play a melody and light up the top row of lights. Turn the bright red knob left, to hear a wacky sound. Turn it right to hear a different wacky sound.

This unit may also be wall mounted with the included mounting hardware.

#### Troubleshooting:

**Problem:** The Therapeutic Manipulator does not operate properly.

**Action #1:** Make sure the batteries are in the battery holder properly and making good contact.

**Action #2:** If the vibration and spinning glitter does not work, replace the C cell batteries.

**Action #3:** If the lights, and wacky sounds do not work replace the AA batteries.

**Action #4:** If does not work at all change all the batteries.

#### Care of Unit:

The Therapeutic Manipulator can be wiped clean with any household multi-purpose cleaner and disinfectant.

**Do not submerge** the unit as it will damage the contents of the top and the electrical components.

Do not use abrasive cleaners, as they will scratch the surface of the unit. Rev 4/17/17

# **Therapeutic ManIpulator #2304**

## **USER'S GUIDE**



### For Technical Support:

Call our Technical Service Department Monday through Friday, 9 a.m. to 5 p.m. (EST) 1-800-832-8697

customer support@enablingdevices.com



50 Broadway Hawthorne, NY 10532 Tel. 914.747.3070 / Fax 914.747.3480 Toll Free 800.832.8697 www.enablingdevices.com