

Fig. 1



Fig. 2



For Technical Support:

Call our Technical Service Department Monday through Friday, 9 a.m. to 5 p.m. (EST) 1-800-832-8697

customer support@enablingdevices.com

Talkable II #2400, III #2401 & IV #2402

(With Built in Icon holder)

USER'S GUIDE



50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

These compact message communicators allow for simple communication, and come with built in icon holders. The Talkable II allows the user to record two messages. Choose between two activities such as asking and answering questions (e.g. yes, no). It has 10 seconds of recording time on each side and two output jacks to activate two devices. The Talkable III allows for three 5-second messages, three jacks allow you simultaneously activates a toy or device for immediate reinforcement, and the Talkable IV allows for four 5 second messages with four jacks to operate devices. All the Talkables come with clear covers. Size for II: $7\frac{1}{2}$ " x $4\frac{1}{2}$ " x 3". Requires 4 AAA Batteries Size for III: $10\frac{1}{2}$ " x 5" x $2\frac{3}{4}$ ". Talkables IV Size: 14"L x $5\frac{1}{2}$ "W x $2\frac{3}{4}$ "H Requires 4 AA Batteries.

Operation:

- 1. Carefully turn unit over and remove the small screw from the battery compartment. Slide the battery compartment cover off. Observing polarity, install four AA size batteries (Not included) into the holder. Replace cover and screw. For Talkable 2 you must install 4 AAA batteries. Alkaline batteries must be used only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly.
- 2. Never mix brands, types or old and new batteries together. Replace the battery compartment cover and replace the screw.
- 3. Turn the unit on by turning the ON/OFF/VOLUME knob located on the bottom panel of the unit. To record, first press and hold the red "RECORD" button press down one of the colored plates. While continuing to hold the plate down, speak your message. Release both when you finish recording. Record on the remaining plates in the same manner.
- 4. To play back your message, press lightly on the plate of your choice. It is not necessary to continue holding the plate down when listening to the message. The unit will continue to playback the entire message, even if pressure on the plate is released.
- 5. The unit will retain recorded information indefinitely, even if the batteries are disconnected. Previously stored information will be erased only when a new recording is performed.
- 6. Playback volume can be adjusted using the "ON/OFF/VOLUME" control knob located on the bottom panel of the unit.

To use as a switch:

7. This unit can also be used to activate switch adapted toys or devices. Simply attach the toy or device to the corresponding 1/8 inch female jack using the cords provided. There is an output jack for each or the

- switch plates. When the plate is pressed, the message will play and the toy or device will be activated. Make sure all connections are tight with no gaps between plug and device.
- 8. The Talkables have two locations for placing Icons or Pictures. On the back of the unit there is a holding strip where Icons or Photos can be placed. There is also a removable clear cover on the built in round switch plate which icons or photos can be placed. When inserting icon in the back holding strip, insert your icon by starting at one side of the strip. Put the icon in at a slight angle for easy installation. See figure on back page.

Please Note: When using the holding strip if you're Icons, Photos or words are printed on regular printer paper, we recommend laminating them. If not they will not stand upright or be secure in the holding strip.

Troubleshooting:

Problem: The unit fails to operate, or operates incorrectly.

Action #1: Make sure there is no dirt or obstacles between the plates and the housing of the unit.

Action #2: Disconnect the power by removing one of the batteries for 20 seconds in order to reset the unit. Re-install the batteries. If the unit does not work correctly, replace all the batteries.

Problem: My toy/device connected to the Talkable does not work.

Action #1: Make sure there are no gaps between your cord connections.

Action#2: Try your toy/device with another switch you know works to rule it out as the source of the problem.

Care of the Unit:

The Talkables can be wiped clean with any mild household multipurpose, non-abrasive cleaner and disinfectant. We recommend Simple Green, which is a non toxic biodegradable all-purpose cleaner.

Do not use abrasive cleaners, as they will scratch the surface of the unit.

Do not submerge the unit, as it will damage the electrical components.