

## Troubleshooting:

**Problem:** The CD does not function.

**Action:**

- Check batteries for polarity placement and be sure to use fresh batteries.
- Keep mobile phones and other electrical appliances far away from the unit. Such appliances may cause noise interference or play may stop entirely.
- Be sure the FUNCTION button is in the position marked "CD".
- Make sure you only have one switch connected to the portable CD at a time.

**Problem:** The Radio does not function.

**Action:**

- Be sure the FUNCTION button is pushed to Radio. If the radio still doesn't function, check that the Channel Tuning knob is in the desired position for reception.

**Note:**

- The AM/FM antenna is the thin cable located in the back of the CD player. To get the best reception, extend the antenna.
- If a CD skips it may need cleaning. Wipe off any fingerprints with a soft cloth.
- Do not get wet. Follow manufacturer's recommendations for cleaning.



**For Technical Support:**

Call our Technical Service Department  
Monday through Friday, 9 a.m. to 5 p.m. (EST)  
1-800-832-8697

[customer\\_support@enablingdevices.com](mailto:customer_support@enablingdevices.com)

---

---

## Portable CD Player/Radio #3414 (NAXA Brand)

---

---

## USER'S GUIDE



50 Broadway  
Hawthorne, NY 10532  
Tel. 914.747.3070 / Fax 914.747.3480  
Toll Free 800.832.8697  
[www.enablingdevices.com](http://www.enablingdevices.com)

### Perfect size!

Our Portable CD Player with AM/FM Radio has been adapted to allow you to activate the CD or radio function with your capability switch. Simply plug your capability switch into the radio jack to play music or plug your switch into the CD jack to play and pause your favorite CD. The radio function requires continuous switch activation to play. This full function top-loading CD play has full range stereo speakers with digital LED displays. Easily fits on wheelchair trays. Size: 9"H x 4½"W x 8"L. Requires Capability Switch and 6 C Batteries. Weight: 3¼ lbs.

**PLEASE NOTE: Styles and brands may vary. AC line cord has been removed and AC power function has been disabled for safety reasons. Battery use only.**

### Operation:

1. The portable CD player requires 6 "C" batteries (not included) to operate. Use alkaline batteries only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. Never mix old and new batteries together or different brands or types together.
2. Install the batteries, with careful attention to the battery polarity. Be sure the + and - poles are facing in the correct directions as shown inside the battery compartment.

### Note:

- Do not mix new batteries with old ones.
- Remove the batteries when you do not plan to use the unit for a month or more.
- Only connect one switch to the Portable. Connecting two switches will disable the CD Boom Box.
- Due to switch adaptation the use of an AC power cord has been disabled for safety reasons. This unit will only work with batteries.

### To operate the CD player:

1. Slide the oblong "FUNCTION" button on the upper left hand side of the unit to the position marked "CD".  
Open the CD compartment by lifting the door located on the top portion of the unit. Insert your CD, label side up, over the center

hub of the disc and gently press the center of the CD until it snaps down. Gently close the compartment door.

2. There are two adapted switch inputs located on the left side of the unit. Insert a capability switch into the outlet marked PLAY/PAUSE "CD". (Input jack only accepts 1/8" switch plugs).
3. When the Capability Switch you have chosen is activated the CD player will turn ON. It will then display the number of tracks that your CD has on it. Press your switch again, and the CD will start to play the first track. Press the switch again to pause the track. (The manufacturer's Play/Pause button will work concurrently.)

### To operate the Radio:

1. For Radio: Manually slide the "FUNCTION" button to "RADIO". Set the "BAND" switch, located on the top/right to AM or FM. Tune into the channel of your choice using the large knob on side of the unit just above the BAND switch. Adjust the volume to your desired listening level.
2. Insert your switch plug into the switch jack located on the left side of the unit marked "RADIO".
3. Insert your switch plug into the switch jack located on the left side of the unit marked "RADIO".
3. If using a momentary switch pressing on the switch will activate the RADIO, releasing will deactivate. Activate your Capability Switch (Only with the use of a switch with latch/timer capability or a switch modifier module like our #605). And the radio will play. Activate it again and it will turn off. **Please Note:** The use of a Latch/Timer switch or switch modifier will not work for the CD portion of this player.

**Please Note:** When finished using the CD player, set the CD/OFF/RADIO selector switch to OFF to prevent battery drain. Failure to do so will result in the batteries draining out.