Adapted Sony Boom Box #3420

USER'S GUIDE



For Technical Support:

Call our Technical Service Department Monday through Friday, 9 a.m. to 5 p.m. (EST) 1-800-832-8697

customer support@enablingdevices.com



50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

High quality!

We've adapted this portable boom box making it accessible to everyone. It features a CD player, cassette player and recorder, AM-FM radio with 30 station pre-sets, a Mega bass sound system, and audio-in for connecting smart phones and MP3 players. Activate your switch once to turn ON the radio, tape or CD player, activate it a second time to turn the selected function OFF. Has earphone jack for quiet listening. (Please note: audio-in is not adapted for switch use.) Size: 12"L x 8"W x $4\frac{1}{2}$ "H. Requires 6 C Batteries. Weight: $4\frac{1}{2}$ lbs.

PLEASE NOTE: Style may vary. AC line cord has been removed for safety reasons.

Operation:

The CD boom box requires 6 "C" batteries (not included) to operate. Install
the batteries, turn the unit over and slide the battery compartment cover off.
Pay careful attention to the battery polarity. Be sure the + and - poles are
facing in the correct directions as shown in the battery compartment. Use
alkaline batteries only (e.g. Duracell or Energizer). Do not use heavy duty or
rechargeable as they will not work.

Note:

- Do not mix new batteries with old ones.
- Remove the batteries when you do not plan to use the unit for a month or more.
- Only connect one switch to the CD Boom Box. Connecting two switches will disable the CD Boom Box.

To operate the CD player:

- 2. First turn the power ON by pushing the POWER button.
- 3. Open the CD compartment by pushing down on the CD compartment lid. Insert your CD, label side up, over the center hub of the disc and gently press the center of the CD until it snaps down. Gently close the compartment door.
- 4. There are two adapted female switch input jacks located on the side of the unit. Insert a capability switch into the jack marked "CD". (Female jack only accepts 1/8"plug) When the Capability Switch you have chosen is activated the CD player will start to PLAY the first track. Press the switch again to PAUSE the track. Pressing your switch a 3rd time will resume PLAY.

To operate the Radio/Cassette Recorder:

5. For Radio: Press the "RADIO" button once for FM, twice for AM. Tune into the station of your choice using the tune buttons. Adjust the volume to your desired listening level.

- 6. Insert the 1/8" plug into the adapted switch jack located on the side of the unit marked "RADIO/TAPE". Activate your Capability Switch and the radio will PLAY. Activate it again and it will turn OFF. If using a momentary switch pressing on the switch will activate the RADIO, releasing will deactivate.
- 7. For TAPE: Press the "TAPE" button. Next press the STOP/EJECT button to open the cassette tape compartment. Insert your audio tape with the open edge facing up and the side you want to play facing you. Push the cassette door to close it. Manually push the button marked "PLAY".
- 8. Activate the Capability Switch and the cassette player will turn ON activate it a second time to STOP the TAPE.

Troubleshooting:

Problem: The CD does not function.

Action:

- Check batteries for polarity placement and be sure to use fresh batteries.
- Keep mobile phones and other electrical appliances far away from the unit.
 Such appliances may cause noise interference or play may stop entirely.
- Be sure the FUNCTION button is in the position marked "CD".
- Make sure you only have one switch connected to the CD Boom Box at a time.

Problem: The cassette recorder is operating but there is no sound. **Action:**

Be sure the VOLUME control button is adjusted to an audible listening level.

Problem: The Radio/Tape player does not function. **Action:**

- Be sure the FUNCTION button is set to TAPE. If the radio still doesn't function, check that the Channel Tuning knob is in the desired position for reception. If the cassette player still does not work, be sure the PAUSE button is not depressed (in the down position). If it is, press it again and it will be in the "up" position. The tape will resume playing.
 - Make sure you only have one switch connected to the CD Boom Box at a time.
 - If a CD skips it may need cleaning. Wipe off any fingerprints with a soft cloth.
 - Do not get wet. Follow manufacturer's recommendations for cleaning.