



Care & Maintenance:

1. Keep the Adapted Bluetooth Speaker clean by wiping it with a slightly damp cloth when needed. It can be wiped clean with any household multi-purpose cleaner and disinfectant. We recommend Simple Green, which is a non-toxic biodegradable all-purpose cleaner.
2. Keep the Adapted Bluetooth Speaker out of direct sunlight and away from any direct heat source.
3. Do not drop the Adapted Bluetooth Speaker on a hard surface and do not expose it to moisture or immerse it in water.



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-832-8697

customer_support@enablingdevices.com

Adapted Bluetooth Speaker #4090

USER'S GUIDE



50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

Easy-touch buttons!

Individuals with mobility issues can control their own music with this adapted Bluetooth speaker. Three 2½" diameter buttons control six essential functions: On/Off; play/pause; volume up/volume down; and track forward/track back. Use sustained pressure for on/off and volume control and a quick touch control play/pause and track changing. Pairs with any Bluetooth-enabled device. Or plays from micro SD/TF card (not included). Includes USB charging cord. Size: 8½ L" x 5W" x 5H". Weight: 1¼ lbs.

Please Note: Computer required for downloading music to Micro/SD/TF card.

Before first use, remove the safety lock tab, by pulling it out. This is used for shipping and storage to prevent accidental activation.

Operation:

1. The Adapted Bluetooth Speaker has a built in rechargeable battery, which is charged through your computer via the included USB cable or using the included cable with any Tablet or Smartphone wall charger for faster charging. You can use the Adapted Bluetooth Speaker during charging. Play time is between 8 and 12 hours at 70% volume.
2. To charge Adapted Bluetooth Speaker, plug it into a USB port on your computer while the computer is on or using the USB cable or a better way to charge it is with a Table or Smartphone wall charger. The LED light will glow red while charging with the power OFF. When charging completes,(approximately 3 to 4 hours) the light will go OFF with the power OFF or glow blue with the power ON.

Modes of Play:

The Adapted Bluetooth Speaker can playback music via Bluetooth, MicroSD/TF, USB/AUX via cable plugged into a PC Press the big Mode button to change modes. You will hear an audio prompt for each mode.

Please Note: FM Mode does not function on this device.

Please Reference the original manufacture instructions for all of the Adapted Bluetooth Speaker functions and features and any additional troubleshooting info.

Please Note: Some features or functions listed in the original manufactures instructions may not work due to switch adaption.

3. To listen to music stored on a MicroSD/TF, insert the card or drive into the appropriately marked slot on the side of the Adapted Bluetooth Speaker. Turn on Adapted Bluetooth Speaker's On/Off switch On. Press the big "Play" switch on the base. Press it again to pause the music. Press the "Track Forward" or "Track Backward" switches to skip between songs. To adjust volume, use the "Vol +" or "Vol -" switches by pressing and holding them down to raise or lower the volume level.
4. When you are finished using the Adapted Bluetooth Speaker make sure to press the big button ON/OFF switch on the base to prevent the battery from draining down all the way. If not and the battery drains out and left in this state could cause damage to the battery.

Troubleshooting:

Problem: Adapted Bluetooth Speaker does not work.

Action #1: Make sure you have charged the Adapted Bluetooth Speaker. Red LED ON: Charge, Red LED OFF: Full Power & Red LED Flashing: Low Power.

Action #2: Make sure that there is nothing inhibiting the depression of the button switches or holding any of the button switches down.

Action #3: Try reloading your music on the MicroSD/TF card, or try a different MicroSD/TF card to rule this out as the issue.