TV Remote Module #5150

USER'S GUIDE



For Technical Support:

Call our Technical Service Department Monday through Friday, 9 a.m. to 5 p.m. (EST) 1-800-832-8697

customer support@enablingdevices.com



50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

Make watching TV easier than ever before!

Just plug a switch or switches into the jacks, and the buttons on the remote will be activated. For example, if your TV remote has a switch plugged into the "channel up" jack, then every time you press the external capability switch, the channel will increase by one. Perfect for viewers with impaired fine motor coordination. Options include power, volume up, volume down, channel up and channel down. Size: $10^{\text{HL}} \times 4^{\text{HW}} \times 2^{\text{HL}}$. Requires Capability Switches and 2 AA Batteries. Remote style may vary. Weight: $\frac{1}{2}$ lb.

Operation:

- 1. Please refer to the manufacturer's instruction manual included for programming directions for this remote, with the exception of Step #1: Installing the Batteries. Please see below:
- 2. The TV Remote Module requires 2 AA batteries (Not included). Use alkaline batteries only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. Never mix old and new batteries together or different brands or types together.
- 3. You will not be placing any batteries directly into the "Sony RMEZ4" remote as this is permanently affixed to the blue base. Instead, please locate the battery compartment recessed in the bottom of the base. The battery compartment cover is secured with a small screw. Use a Phillips head screwdriver to remove this screw and remove the cover. Insert the 2 AA batteries into the compartment paying careful attention to proper battery polarity.
- 4. Replace the compartment cover and re-secure it with the screw. Test the unit by pressing the ON/OFF button. The red indicator light should blink. If not, change your batteries.
- 5. When programming is complete, plug in an external capability switch and check for successful programming by pointing the remote in the direction of the TV and activating the switch.
- 6. The TV Remote Control Module is equipped with 1/8" jacks for use with capability switches with 1/8" plugs. Make sure that the connections between the external capability switches and the jacks are tight. There should be NO gaps. Even slight gaps could cause a malfunction.
- 7. If it fails to operate, please note that your TV brand may have several codes listed in the table. Try each of them until you find one that works.

Troubleshooting:

If you cannot get your TV Remote Module to operate you're TV, or it will not perform commands properly:

- Make sure that your batteries are fresh and installed correctly (according to proper polarity). *
- Make sure your switches are plugged in all the way, there should be no gaps.
 Test your switches on other switch adapted devices, to rule out your switches as the source of the problem.
- Read the" Sony's" troubleshooting section of the guide for any additional troubleshooting tips.
- Try <u>ALL</u> of the codes listed for your TV. Many set-up codes may allow partial functionality; however, only one code will operate your TV correctly.
- Make sure that you are within 6-10 feet of your device, with the remote facing the TV, while programming. Make sure that you are within 15 feet of your TV when using your remote.
- If the red light on the remote control does not blink when you press a key, replace the batteries with two (2) new AA alkaline batteries. Be sure to install paying close attention to proper battery polarity. *.
- NOTE: When you replace your batteries, or remove them for any reason, you
 must re-install batteries within 10 minuets, or re-programming your remote will
 be required. This is why it is important to write down the codes that work for
 your TV and keep both these manuals in a safe place.

Care of Unit:

The TV Remote Module can be wiped clean with any household multi-purpose cleaner and disinfectant. We recommend Simple Green, which is a non toxic biodegradable all-purpose cleaner.

Do not submerge the unit, as it will damage the contents and the electrical components.

Do not use abrasive cleaners, as they will scratch the surface of the unit.