

Textured Plate Switches #7225B (Blue), 7225O (Orange), 7225R (Red), 7225Y (Yellow) (Set of 4) #7226

USER'S GUIDE

Enabling For Technical Support: Call our Technical Service Department Monday through Friday, 9 a.m. to 5 p.m. (EST) 1-800-832-8697

customer_support@enablingdevices.com



50 Broadway Hawthorne, NY 10532 Tel. 914.747.3070 / Fax 914.747.3480 Toll Free 800.832.8697 www.enablingdevices.com Light-touch tactile switches activate with the slightest movement. Four textures in four colors to choose from: blue with inverse bumps, orange with thin ridges, red with wide ridges, and yellow with bumps. Buy all four and save. Size: 4"L x 6"W x $1\frac{1}{2}$ "H. Weight: $\frac{1}{2}$ lb.

Operation:

- 1. The Textured Switch does *not* require any batteries to operate.
- 2. Plug the Textured Switch into the jack on your switch-adapted toy/device.
- 3. If it is necessary for you to use an adapter, make sure that you use a MONO adapter, not a STEREO adapter.
- 4. Make sure that the connection between the Textured Switch and your toy/device is plugged in all the way. There should be no gaps.
- 5. Simply press anywhere on the Textured Switch's textured colored top to activate your toy/device.
- 6. The toy/device will only remain activated while the Textured Switch continues to be actuated. Once you release the switch, the toy/device will turn off.

Troubleshooting:

Problem: The Textured Switch does not activate your toy/device.

Action #1: Make sure that the connection between the Textured Switch and your toy/device is plugged in all the way. There should be *no* gaps. This is a common error and an easy fix.

Action #2: Try a different switch with your toy/device to rule out the toy/device as the source of the problem.

Action #3: Try a different adapter (if applicable) to rule this out as the source of the problem.

Care of Unit:

The Textured Switch can be wiped clean with any household multi-purpose cleaner and disinfectant. We recommend Simple Green, which is a nontoxic biodegradable all-purpose cleaner.

Do not submerge the unit, as it will damage the contents and the electrical components.

Do not use abrasive cleaners, as they will scratch the surface of the unit.