
Musical Fan Switch #742 & 742B

USER'S GUIDE



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-832-8697
customer_support@enablingdevices.com



50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

A great cause and effect switch!

Press the delightfully textured oval plate and feel, hear and see the fan turn on. Connect your device to this switch and it will simultaneously turn on your device. Size: 9¼"L x 6½"W x 6¼"H. Requires 4 AA Batteries. Weight: 1¼ lbs.

Operation:

1. The Musical Fan Switch requires four AA batteries to operate. Use alkaline batteries only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. Never mix old and new batteries together or different brands or types together.
2. Carefully turn over unit to reveal battery compartments. Remove the Phillips Head screws from the battery compartment covers. Next slide the battery compartment covers off and insert two "AA" batteries into each of the holders. Be careful to check for Polarity, matching + pole to + marking and - pole to - marking in the battery holders. Replace the covers and screws.
3. Set the ON/OFF switch to ON. Press anywhere on the textured yellow switch plate and you will be rewarded with a gentle breeze, lights and music. Once you release pressure on the textured yellow plate, the fan, lights and music will stop.
4. To use the Musical Fan as a switch, attach one end of the double ended 1/8" to 1/8" male cord (included) into the back of the Musical Fan, via the 1/8" female jack. Next plug the other end of the cord into your adapted toy or device. When the yellow Plate is pressed the toy or device is activated simultaneously. Release the plate and your toy/device will turn off.

Caution:

Keep loose hair away from fan to prevent tangling in fan blades.

5. When you are finished using the Musical Fan switch turn the unit OFF. This will prevent battery drain. If the Musical Fan switch is not going to be used for a long period of time, e.g. one month or longer, we recommend removing the batteries during storage to prevent battery drain or leakage, which can cause damage to the Fan.

Troubleshooting:

Problem: Musical Fan does not work.

Action #1: Check the batteries for proper installation, as noted in operation No. 2.

Action #2: Check batteries for freshness, if weak or dead replace with new batteries.

Action #3: Make sure there is nothing obstructing the fan blades or the textured yellow switch plate.

Action #4: When used as a switch, make sure all connections between Musical Fan and your Toy/Device are tight. There should be no gaps.

Action #5: Check your Toy/Device with another switch to rule the Musical Fan out as the source of the problem.

Care of Unit:

The Musical Fan switch can be wiped clean with any household multi-purpose cleaner and disinfectant. We recommend Simple Green, which is a non toxic biodegradable all-purpose cleaner.

Do not submerge the unit, as it will damage the contents and the electrical components.

Do not use abrasive cleaners, as they will scratch the surface of the unit.