# High Roller #757

## **USER'S GUIDE**



### For Technical Support:

Call our Technical Service Department Monday through Friday, 9 a.m. to 5 p.m. (EST) 1-800-832-8697

customer support@enablingdevices.com



50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

#### Shake, rattle and roll!

We've created an adapted dice roller so everyone can join in on the fun. Simply open the Velcro top and place in up to five dice. Push the outer rim or add your capability switch to activate the dice. The dice will shake, rattle and roll! Use High Roller for learning number recognition. Comes with two sets of dice, one white and one black. Size: 6"L x 6"W x 3¾"H. Requires 2 AA Batteries. Weight: ¾ lb.

#### Operation:

- The High Roller requires two AA batteries. Use alkaline batteries only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. Never mix old and new batteries together or different brands or types of batteries together.
- 2. Gently turn the unit over to reveal the battery compartment. Carefully remove the small screw from the battery compartment cover using a Phillips head screwdriver, and slide the cover off. Observing proper (+) & (-) battery polarity, install two AA size batteries into the holder. Replace the battery compartment cover and screw. Press the push button ON/OFF switch to turn the High Roller ON.
- To place your dice in the High Roller, open the clear dome by lifting up on the Velcro tab. Place up to 5 dice into the tray then close and secure the dome with the Velcro tab. Now you're ready to play your favorite dice game.
- 4. The High Roller can be activated two ways. First pressing down on the clear dome or outer rim. You can also activate the High Roller through the use of a capability switch (not included). Plug your switch into the 1/8" jack located on the side of High Roller. Make sure there are no gaps in the connection. Activating your switch or pressing on the clear dome/outer rim of the High Roller will make the dice shake. Once you release your switch or stop pressing down on the clear dome/outer rim, the High Roller will stop shaking the dice.

**Please Note:** When finished using the High Roller, press the ON/OFF switch to OFF for storage. We recommend removing the batteries when storing for long periods of time.

### **Troubleshooting:**

**Problem:** The High Roller does not shake the dice.

Action # 1: Make sure the High Roller's ON/OFF switch is set TO on.

**Action # 2:** Rotate (spin) the batteries within the battery compartment to ensure a good connection. Check that all batteries are installed properly.

**Action # 3:** Check all batteries for freshness, if weak or dead replace with new batteries. Use an alkaline battery (E.G. Duracell or Energizer brand).

**Action # 4:** Make sure the High Roller is on a flat smooth and level surface. Any obstructions under the base will prevent you from activating the High Roller.

**Action # 5:** Make sure that you have a tight connection between the external switch and the High Roller. There should be no gaps.

**Action # 6:** Try your external capability switch with another toy/device to rule out the switch as the source of the problem.

#### Care of the Unit:

The High Roller can be wiped clean with any mild household multipurpose, non-abrasive cleaner and disinfectant. We recommend Simple Green, which is a non-toxic biodegradable all-purpose cleaner.

Do not use abrasive cleaners, as they will scratch the surface of the unit.

Do not submerge the unit, as it will damage the electrical components.