



enabling
devices

For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-832-8697
customer_support@enablingdevices.com

Adjustable Angled Switch/Communicator #806

USER'S GUIDE

enabling
devices

50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

Bright lights!

Our sequencer is designed for the visually impaired. The circular patterned red LED lights in the center give the user a visual cue for targeting and encourage the user to activate the switch. The angle of the big 5" switch is adjustable from 32° to 90°. The light can be turned on or off. This sequencer has a total of 300 seconds recording time and can be clamped. Also works as a switch. Easy to mount on tabletops with a clamp. (C-clamp sold separately.) Size: 8"L x 8"W x 6¼"H. Requires 4 AA Batteries. Weight: 1¼ lbs.

Operation:

1. Adjustable Angled switch Communicator requires four AA batteries. Use alkaline batteries only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. Never mix old and new batteries together or different brands or types together.
2. Gently turn the unit over to reveal the black battery compartments. Carefully remove the small screws from each battery compartment cover using a Phillips head screwdriver, and slide the covers off. Observing proper (+) & (-) battery polarity, install four AA size batteries into the holders. Replace the battery compartment covers and screws.
3. There are two controls between the battery compartments; one is an ON/OFF switch for the LED lights on the front of the switch. When turned ON, the LED lights will stay ON until this switch is turned OFF. Please Note: when finished using the switch turn the LED's OFF to prevent battery drain. The second control is the ON/OFF/Volume knob. This is used when the switch will be used as a communicator.
4. To make a Recording, turn the ON/OFF/VOLUME knob ON you will hear a slight click, then adjust the volume to an audible level. **Please Note: when recording make sure the LED lights are turned OFF.** The **Microphone** is located on the left side of the unit. To record, flip the **RECORD/PLAY** switch to the **RECORD** position. Depress the Yellow switch plate and speak 6 to 8 inches from the microphone. When you are finished recording the first message, release the Yellow switch plate. To record all subsequent messages, simply depress the switch plate, speak into the microphone, releasing the switch plate after each message.
5. Messages can vary in length and the total record time is 300 seconds. To play the messages, flip the **RECORD/PLAY** switch to the **PLAY** position. Activation of the Yellow switch plate will play the first message; the second activation will play the second message, and so on. To interrupt the messages and start at the beginning of the sequence, press the **RESET** button on the back of the unit.
6. The unit will retain recorded information indefinitely, even if the battery is disconnected. Previously stored information will be erased only when new recording is performed.
7. **Please Note:** When re-recording messages, all messages on the unit will be erased.

8. To turn the unit off, turn the volume control knob counter-clockwise until you hear an audible "click".
9. The Adjustable Angled switch can be adjusted from 20 to 90 degrees by simply losing the two knobs on the side of the unit and pulling up the big 5" Yellow Plate to the desired angle. Once you have the desired angle simply tighten the two set knobs to lock in the angle. Do not over tighten the set knobs. Finger tight is all that is needed. (See photo on back cover.)
10. All it takes is a slight push on the plate, a roll up action by your hand or a swiping action to activate the switch. There are also three different spots on the black base mold to clamp the switch to a table top or wheelchair tray.
11. To use the switch, attach one end of the double ended 1/8" to 1/8" male cord (included) into the back of the Adjustable Angled switch, via the 1/8" jack. Next plug the other end of the cord into your adapted toy or device. Make sure all connections are tight with no gaps. When the yellow Plate is pressed the toy or device will activate. Release the plate and your toy/device will turn off.

Troubleshooting:

Problem: My toy/device does not work with the Adjustable Angled switch.

Action #1: Check the batteries for proper installation in your toy/device.

Action #2: Check batteries for freshness, if weak or dead replace with new batteries.

Action #3: Make sure there is nothing obstructing the yellow switch plate on the Adjustable switch.

Action #4: Make sure all connections between Adjustable Angled switch and your toy/device are tight. There should be no gaps. If using an adapter it must be a mono adapter, not stereo.

Action #5: Check your toy/device with another switch to rule the Adjustable Angled switch out as the source of the problem.

Care of Unit:

The Adjustable Angled switch can be wiped clean with any household multi-purpose cleaner and disinfectant. We recommend Simple Green, which is a non toxic biodegradable all-purpose cleaner.

Do not submerge the unit, as it will damage the contents and the electrical components.

Do not use abrasive cleaners, as they will scratch the surface of the unit.