Switch & Glow #894

USER'S GUIDE



For Technical Support:

Call our Technical Service Department Monday through Friday, 9 a.m. to 5 p.m. (EST) 1-800-832-8697

customer support@enablingdevices.com



50 Broadway Hawthorne, NY 10532 Tel. 914.747.3070 / Fax 914.747.3480 Toll Free 800.832.8697 www.enablingdevices.com

Oodles of fun!

Our neon stretchy noodle switch provides you with wonderful tactile and visual experience. The lightest touch on the switch turns on a glowing light while activating any toy or device. Size: $7"H \times 6\frac{1}{2}"W \times 7"D$. Requires 2 AA Batteries. Weight: 1 lb. PLEASE NOTE: This product is not suitable for those who are allergic to latex.

Operation:

- 1. The Switch & Glow requires 3 AA batteries to operate (Not included). Use alkaline batteries only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. Never mix old and new batteries together or different brands or types together.
- 2. Gently turn the unit over to face the black battery compartment cover. Carefully remove the small screw from the battery compartment cover using a Phillips head screwdriver, and lift the cover off, it may be necessary to use the end of the screwdriver to lift one edge of the cover. Observing proper (+) & (-) battery polarity, install 3 AA size batteries into the holder. Replace the battery compartment cover and screw.
- 3. Turn the ON/OFF switch to ON.
- 4. Plug the Switch & Glow into the jack on your switch-adapted toy/device. Make sure that the connection between the Switch & Glow and your toy/device is tight. There should be no gaps.
- 5. Simply press anywhere on the Switch & Glow's squishy Neon colored top to activate its illumination and your connected toy/device.
- 6. The illumination and toy/device will only remain activated while the Switch & Glow Switch continues to be actuated. Once you release the switch, the illumination and toy/device will turn OFF. The Switch & Glow Switch can be used as a sensory stimulating toy on its own. You can also use it as a switch without the illumination.

Important Notes: Never use the Switch & Glow without the stretchy cover, the light inside is very bright and should never be looked at directly.

For prolonged storage we recommend removing the batteries.

This switch will not work if the batteries are removed, weak or dead.

Batteries are needed for switch to work.

Troubleshooting:

Problem: The Switch & Glow Switch does not activate your toy/device.

Action #1: Make sure that the connection between the Switch & Glow Switch and your toy/device is tight. There should be *no* gaps. This is a common error and an easy fix.

Action #2: Try a different switch with your toy/device to rule out the toy/device as the source of the problem.

Action #3: Try a different adapter (if applicable) to rule this out as the source of the problem.

Action #4: Make sure the batteries are fresh; this switch will not work without batteries.

Care of Unit:

The Switch & Glow Switch can be wiped clean with any household multipurpose cleaner and disinfectant. We recommend Simple Green, which is a non toxic biodegradable all-purpose cleaner.

Do not submerge the unit, as it will damage the contents and the electrical components.

Do not use abrasive cleaners, as they will scratch the surface of the unit.