

Easy Flex Stylus - #9079& Replacement Tips #9091

USER'S GUIDE



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-832-8697

customer_support@enablingdevices.com



50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

Our most versatile and adjustable stylus allows you to position it in just about any position – between fingers, above your hand, under your hand, around your wrist. Whatever works best for you! Comes with two adjustable Velcro straps for stabilizing and securing the stylus. Full unadjusted size: 14"L x 4"W. Weight: ¼ lb.

Operation:

1. The Easy Flex Stylus comes with an adjustable Velcro strap, the Easy Flex Stylus can be positioned between any finger and the strap can be used if needed. Once you have the position set you can now secure the Easy Flex Stylus with the Velcro strap (if needed) across the hand as shown below.



2. The Easy Flex Stylus is a flexible rod you can adjust by bending to fit your needs.

Replacement Tips:

When it comes time to replace the Stylus tip you will first need to remove band of silver tape wrapped around the base of the Stylus. Next you will unscrew the tip and replace it with one of the replacement tips (sold separately). The replacement tips will include a new tape strip to re-apply to secure the tip to the Stylus. **Please Note:** The tape must be used to prevent the tip from becoming a choking hazard for young children.

Care of Unit:

The Easy Flex Stylus can be wiped clean with any mild household multi-purpose cleaner and disinfectant.

Do not submerge the unit, as it will damage the contents of the top and the electrical components.