



Photo 4

enabling
devices

For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)

1-800-832-8697

customer_support@enablingdevices.com

**Adjustable Touch Screen Head Stylus
#9092
Replacement Tips #9091**

USER'S GUIDE

enabling
devices

50 Broadway

Hawthorne, NY 10532

Tel. 914.747.3070 / Fax 914.747.3480

Toll Free 800.832.8697

www.enablingdevices.com

Tablet access!

An essential aid for people with limited hand use. Our telescoping stylus is mounted on a light and comfortable headpiece and works with the slightest touch on your tablet, smartphone or computer. *Headpiece adjustable*. One size fits all; fits directly on head

- Adjustable nylon straps with non-slip rubber inserts; polycarbonate buckles
- Measures 6 to 7 inches in diameter; stretches to 12 inches in diameter; Stylus adjusts from 9" to 20"L. Weight: 6 oz.

Operation:

1. The Adjustable Head Stylus is adjustable for head sizes 6" to 12" in diameter. The adjustments are made on the side and top of the headband. Adjust to a snug but comfortable fit. Once you have the adjustments done for a good fit (shown in photo 1), you can adjust the telescoping stylus for use.



Photo 1

2. The Adjustable Stylus is on a telescopic rod you can adjust its length to fit your needs. The stylus will adjust from 9" to 21½" inches. You are also able to adjust the angle of the telescopic rod up and down. Adjustment is made by first loosening the Philips Head screw on the Headband mounting base. Then the telescopic rod can be adjusted up or down to accommodate the user. Once the desired angle is set, lock it in place by tightening the Philips Head screw. Caution, do not over tighten the screw.



Photo 2



Photo 3

Replacement Tips:

When it comes time to replace the Stylus tip you will first need to remove band of silver tape wrapped around the base of the Stylus. Next you will unscrew the tip and replace it with one of the replacement tips (#9091 sold separately). The replacement tips will include a new tape strip to re-apply to secure the tip to the Stylus. **Please Note:** The tape must be used to prevent the tip from becoming a choking hazard for young children. See Photo 4 on back page of this guide.

Care of Unit:

The Adjustable Stylus can be wiped clean with any mild household multi-purpose cleaner and disinfectant.

Do not submerge the unit, as it will damage the contents of the top and the electrical components.