## **EZ Mount Tray & Optional Adapters**

1594 Small Talk EZ Mount Tray 1595 EZ Mount Tray Adapters 1596 Big Talk EZ Mount Tray 1597 Talkable 3 EZ Mount Tray 1598 Talkable 2 EZ Mount Tray 1601 Bright Red Switch EZ Mount 1602 iPad Wireless Switch EZ Mount Tray





50 Broadway
Hawthorne, NY 10532
Tel. 914-478-0960 / Fax 914-478-7030
www.enablingdevices.com

"A game changer"—that's what our resident Occupational Therapist calls this new mounting system for our Big Talk, Talkable 2 and Talkable 3 communicators. Originally designed for a customer whose wheelchair was overloaded with traditional mounts, the EZ Mount system is now available to everyone at an affordable price.

How it works: For a fast and easy mounting solution, all you need is the EZ Mount Tray for your Big Talk, Talkable 2 or Talkable 3 communicator. Snap the communicator into the tray, wrap the Velcro<sup>TM</sup> strap around the arm of a chair and you're done. For more versatility, the EZ Mount Tray Adapters (#1595, see our website) gives you three additional mounting options:

- 1. A mounting adapter for tubular rails of 1", 7/8", 3/4", or 5/8" in diameter;
- 2. An adapter that integrates with a gooseneck that has a  $\frac{1}{4}$ -20 female threaded socket; and
- 3. An adapter that is compatible with all of our current Magic Arm mounts with a 5/8-inch diameter post.

**NOTE:** The EZ Mount Trays only fit the latest generation of Big Talks, Talkable 2s and Talkable 3s manufactured in 2022. If you are unsure which device you have, please compare yours to the images here or call us at 1-800-832-8697 for assistance.

## **OPERATION:**

All EZ Mount trays work in the same way. The photos below show (#1594 Small Talk & #1596 Big Talk EZ Mount Tray), but the operation is the same for other trays. The included Velcro mounting strap can be used to mount the device tray on a chair arm, wheelchair walker or bed rail to name a few. To use, slip the Velcro strap through the two slots in the bottom of the tray. Next press your device into the tray to lock it in place. Finally, slip the one end of the strap through the loop and tighten until secure. See photos below.



Page #1

The set includes EZ Mount Tray Adapters for use with all EZ Mount Trays:

- 1. An adapter for tubular rails of 1", 7/8", 3/4", or 5/8" in diameter;
- 2. An adapter that integrates with a gooseneck that has a ½-20 female threaded socket; and
- 3. An adapter that is compatible with all of our current Magic Arm mounts with a 5/8" diameter post.

Option 1 is for mounting to a tubular surface, such as a bed rail. From the included, color-coded inserts, select the correct insert for your size tubing. The size is indicated on the insert. Next, attach the mounting adapter to your device's mounting tray with the two included Phillips head screws. Place the base of the mounting adapter and insert over your tubing. Now place the upper half of the mounting adapter through the two screws and secure with the two black thumbscrews. Do not over tighten. See photos.





Option 2 is for mounting to a gooseneck mounting system with a  $\frac{1}{4}$ -20 threaded female socket. Attach your device's mounting tray to the adapter and secure with the two Phillips head screws. Thread the adapter with the  $\frac{1}{4}$ -20 screw into your gooseneck. Place your device into the mounting tray and press down to lock in place. See photos.



The last mounting option is for mounting to any mount with a 5/8" diameter post, such as our Light Duty Magic Arm Mounting System #1557 and our Magic Arm Mounting System #1631. First, attach the mounting adapter with the 5/8" diameter hole and single adjustable thumbscrew to your device's mounting tray with the two Phillips head screws. Next, line up the thumbscrew with the flat spot on the mounting arm's post. You will need to back the thumbscrew almost all the way out to allow the block to seat all the way down on the post. Once this is done, tighten the thumbscrew to secure the mounting plate to your mounting arm. Place your device into the mounting tray and press down to secure. See photos below.



## **Troubleshooting:**

**Problem:** Mounting tray is loose on the mounting Adapter.

**Action:** Make sure that you have secured the mounting tray to the adapter with the included two Philips head screws, and that they are tightened properly.

**Problem:** Mounting adapter is loose on mounting arm.

**Action #1:** Make sure you have tightened the  $\frac{1}{4}$ -20 post all the way to your gooseneck.

**Action #2:** Make sure the thumbscrew, is lined up with the flat spot on your 5/8" mounting post. Second, make sure the thumbscrew is tightened all the way.

**Problem:** My mounting tray adapter is loose on my tubular mount (e.g., bed rail).

**Action #1:** Make sure you have selected the proper size insert needed for your tubing's diameter.

**Action #2:** Make sure both thumbscrews are tightened properly.

## Care of Unit:

The mounting tray and adapters can be wiped clean with any household multi-purpose cleaner and disinfectant. We recommend Simple Green, which is a non-toxic, biodegradable all-purpose cleaner.

**Do not use abrasive cleaners**, as they will scratch the surface of the unit.

 If you have tried all of the above and are still having problems, contact our Tech Support Department at 1-800-832-8697 ext. 320 or via email at: <u>customer\_support@enablingdevices.com</u>